

December 19, 2008

Dear customer,

RE: SOME HELPFUL TIPS WHEN USING THE EPOTEK 302-3M ADHESIVE FOR ASSEMBLY OF THE ASTICS[®] RETAINER

- 1) Always mix part A and part B separately in their containers before mixing them together.
- 2) At least 3 grams of the material should be weighed out each time the product is mixed.
- 3) A proper calibrated scale should be used for weighing (e.g. triple beam balance).
- 4) Disposable polypropylene transfer pipettes may be used for measuring; wooden tongue depressors may be used for mixing.
- 5) Clipping the tip of the pipette (slant cut) may help with dispensing.
- 6) A smaller tip and narrower bulb may give greater control over dispensing.
- 7) The product can be mixed in a small and shallow concave glass recipient, allowing adequate depth for dipping an applicator tip.

If you have any questions about adhesive use please feel free to contact the company by email (contact@biomersbraces.com).

Best regards,

Product Development Team