

## **INSTRUCTIONS FOR USE**

### **Indications for Use**

The ASTICS retainer is indicated for patients that require (1) tooth stabilization after receiving treatment with orthodontic braces, or (2) minor labiolingual tooth movement.

### **Contraindications**

The ASTICS retainer is contraindicated for patients known to have (1) sensitivity to nickel, and (2) active dental disease.

### **Time Duration**

The end-user should wear the appliance for several hours to less than twenty four hours in a given day, and not to exceed 30 days total duration in time on a continuous basis.

### **Warning**

None.

### **Precautions**

Measures to be observed when storing the uncured ASTICS wire include:

- (1) Keep away from light.
- (2) Store at temperatures 4 °C / 39 °F or below.

Measures to be observed when handling the uncured ASTICS wire include:

- (1) Do not make sharp bends.
- (2) Limit prolonged exposure to light.
- (3) Cut only with a sharp edged cutter.
- (4) Do not use sharp tips and or edges to conform to teeth surfaces.
- (5) Do not attempt removal of the hardened sticky wax after the first round of light curing.

Measures to be observed when assembling the ASTICS retainer include:

- (1) The cured ASTICS wire should not be cut. Instead, grind gently using fine grade silicon carbide paper to make minor adjustments.
- (2) Avoid scratching or committing damage to the cured ASTICS wire surface after sheath removal.
- (3) Do not apply excessive force on the hollow section of the metal connector.

- (4) All soldering and polishing must be complete before insertion of the cured ASTICS wire.
- (5) The selected adhesive should not have a curing temperature of more than 100 °C / 212 °F.
- (6) Always remove left over pumice (if any) from the hollow ends of the metal connectors after polishing of the baseplate is complete.
- (7) Do not immediately remove the finished appliance from the working model after curing. Let the appliance cool in temperature for approximately 20 minutes before removal.

**Caution**

Federal law restricts this device to sale by or on the order of an orthodontic professional.

**Manufactured for:**

BioMers Products, LLC  
3425 Radio Road, Unit 204  
Naples, Florida, 34104  
USA

**Manufactured by:**

BioMers Pte. Ltd.  
18 Boon Lay Way, #05 -99  
Singapore, 609966  
Singapore

# Label Icon Legend



Manufacturer  
Fabricant  
Fabricante  
Hersteller  
Fabricante  
Fabbricante  
De fabrikant  
Κατασκευαστής




Authorized Representative in the European Union  
Le Représentant autorisé dans l'Union européenne  
El Representante autorizado en la Unión Europea  
Ermächtigten Vertreter in der Europäischen Union  
Representante autorizado na União Europeia  
Il Rappresentante autorizzato nell'Unione Europea  
Gemachtigde Vertegenwoordiger in de Europese Unie  
Εξουσιοδοτημένος Αντιπρόσωπος στην Ευρωπαϊκή Ένωση

**REF**

Catalogue Number  
Cataloguer le Nombre  
Cataloge el Número  
Katalogisieren Sie Zahl  
Catalogue Número  
Catalogare il Numero  
Catalogiseer het Nummer  
Αριθμός καταλογού



Lot Number  
Nombre de lot  
Número de terreno  
Mengenahl  
Lote Número  
Numero di lotto  
Het perceel Nummer  
Αριθμός παρτίδας

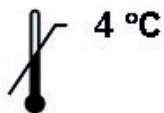

Upper limit of temperature 4 °C  
La limite supérieure de température  
El límite superior de la temperatura  
Obergrenze der Temperatur  
O limite máximo de temperatura  
Il limite superiore di temperatura  
De hogere limiet van temperatuur  
Διατηρείται σε θερμοκρασία μέχρι



Consult instructions for use  
Consulter des instructions pour l'usage  
Consulte las instrucciones para el uso  
Konsultieren Sie Anweisungen zum Gebrauch  
Consulte instruções para uso  
Consultare le istruzioni per l'uso  
Raadpleeg instructies voor gebruik  
Δείτε οδηγίες χρήσης



Keep away from light  
Eviter la lumière  
Aleje de la luz  
Halten Sie fern von Licht  
Mantenha longe de luz  
Stare lontano dalla luce  
Hou van licht weg  
Διατηρείται σε σκοτεινό μέρος



Use by date  
Utiliser par la date  
Utilice por la fecha  
Benutzen Sie bis Datum  
Use por data  
Usare dalla data  
Gebruik tegen datum  
Ημερομηνία λήξης



**EMERGO EUROPE**  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands  
Tel: +31 (0)70 345 8570  
Fax: +31 (0)70 346 7299